CONDOMS FOR YOUR SIZE

Karex Industries Sdn. Bhd.
PTD. 7906 & 7907,
Taman Pontian Jaya,
Batu 34, Jalan Johor,
82000 Pontian,
Johor Darul Takzim, Malaysia.

Advena Ltd.
Tower Business Centre,
2nd Flr., Tower Street, Swatar,
BKR 4013 Malta.

0123 EN ISO 4074

REVISION DATE:
MAY 2020 (V.1)

MASTER REFERENCE INFORMATION:
K-IFU-01 (V.1)

EN Additional Information Condoms:

1. For contraception and prevention of sexually transmitted infections. 2. Handle carefully, when removing from package avoid damage by fingernails, jewellery etc. 3. Do not use the condom if the rubber material is sticky or brittle or the individual container is obviously damaged. 4. If being damaged replace an individual container by a new one and discard the old one. 5. Ask a doctor or pharmacist about compatibility of any topical medicines in possible contact with condom. 6. If extra lubrication is needed, use water-based lubricants and never oily ones like petroleum jelly, baby oil, body lotion, massage oil, butter, margarine usw. 7. Place condom on erect penis before any contact between penis and partner body occurs to assist preventing STIs or pregnancy. 8. Stop if you feel condom slipping off or tightening excessively onto penis because of breakage danger. 9. Withdraw penis soon after ejaculation by holding condom in place at penis base. 10. Should a condom burst or leak during use seek medical assistance as soon as possible, at least within 72 h. 11. Dispose used condom into closed rubbish bin, not into toilet. 12. Don't use twice, reuse might increase risk of failure or infection. 13. Normal condoms are for vaginal use only, not anal sex. 14. NR-Latex might cause allergic reaction incl. anaphylactic shock. 15. **LOOVARA** condoms are a reliable protection, but no contraceptive can protect 100% against pregnancy, AIDS and/or infectious diseases. 16. Store in a cool dry place away from direct sunlight. Keep away from children. Can choke or suffocate on uninflated or broken condom fragments. 17. Use of condom during oral sex increase the risk of asphyxiation or choking. 18. Condom is a medical device to be used by lay persons - adult male or female. 19. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. 20. These condoms conform to International standards ISO 4074/ EN ISO 4074.

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LOOVARA Intimate GmbH,