nevernot information

**DON'T FLUSH
USE ONCE ONLY**

nevernot.de #nevernot @ nevernot_official

Do what you want

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EN nevernot tampon with or without withdrawal cord

nevernot soft tampons allow you to have pleasant and hygienic sex during your period. Also, due to the lack of a string and their super soft, flexible texture they are the ideal menstrual product for sports, swimming, sauna or beauty treatments in the intimate area (such as waxing). The unique shape makes nevernot soft tampons especially easy to remove.

One size fits all:

- suitable for all menstruating users
- regular absorbency

How to insert:

1. Wash your hands thoroughly before opening the packaging of the tampon.
2. If you prefer to use the soft tampon with a withdrawal cord, just pull the cord through the finger loop and fasten it with a knot so that the cord cannot come off the soft tampon. When using soft tampons for the first time, we always recommend using them with a cord attached until you get used to removing them without the string. Otherwise, just dispose of the withdrawal cord via your general waste.
3. Applying a dab of lubricant or clean water to the soft tampon can make it easier to insert.
4. Hold the soft tampon between your thumb and middle finger with the finger loop pointing downwards and squeeze it. The soft and flexible material adapts to every female body and makes it easy to insert, even though soft tampons may appear slightly larger than usual tampons.
5. Now you can first insert the tampon into the vagina with your thumb and middle finger and then push it further up in front of your cervix using your index or middle finger. You can easily insert the tampon standing up, with one leg raised or in a squatting position.

Duration of use:

The duration of use depends on your individual blood loss (roughly 2-4 hours during heavy flow, 4-6 hours during lighter flow). After sexual intercourse or swimming you should immediately change the tampon in order to reduce the chance of an infection. To guarantee safe use of the product, the duration of use must not exceed 8 hours. Please always remember to remove the previously inserted tampon before inserting a new one and do not forget to remove the last tampon at the end of your period.

How to remove:

1. Wash your hands thoroughly before removing the tampon.
2. You can easily pull out the soft tampon using the withdrawal string, but it can also be removed without a thread. To do this, simply follow the steps below.
3. Most of the time, soft tampons can be easily removed from the vagina while sitting on the toilet and using your thumb and index finger or index and middle finger (pinch grasp). Alternatively, you can pull the tampon out by sticking your index or middle finger through the loop at the wider end of the tampon and pulling it out.
4. For easier removal (especially, if the soft tampon has moved further up your vagina during sexual intercourse), we recommend to come into a wide and deep squat - either on top of a towel or in the shower to avoid staining. This position will tilt your pelvis and will shorten the depth of your vagina, making it easier to grab the tampon with your fingers.
5. Also, soft tampons are best removed, when they are fully soaked. If this is not the case, you may use a vaginal douche to add some liquid. Alternatively, a hot bath may help you to further relax and soak the tampon.
6. Please note that every female body is different. For some of us, soft tampons are easier to remove, others find it more difficult. Either way, it is important to know that a soft tampon can not "disappear" in your body. The opening of the cervix is far too small as for a tampon to enter your uterus. Generally speaking: the more you relax, the easier it will be for you to remove the soft tampon and after a few uses, you will have got the hang of it. In case you should really be unable to remove the tampon yourself, please consult your gynaecologist.
7. Please dispose of any used soft tampons through the domestic waste - do not flush!

IMPORTANT!! Please read the following information about Toxic Shock Syndrome (TSS):

TSS is a rare but serious and sometimes life-threatening condition which is associated with the use of tampons. Symptoms may occur suddenly and resemble those of a common flu: temperature of 39°C (102 °F) or more, nausea, diarrhoea, fainting, dizziness and/or rash (similar to sunburn). Treatment: An early diagnosis and treatment is essential in order to prevent a grave progress of the disease. If you suffer from any of the above symptoms, immediately remove your tampon and seek medical assistance. It is crucial to tell your doctor that you have your period and that you have been using a tampon.

At-risk groups and avoidance of the disease: TSS occurs in all humans, however, it comparably often affects women using tampons for menstrual hygiene. Those under the age of 30 have an additional risk to fall ill with TSS. In order to reduce the risk of TSS, we recommend to use tampons with the lowest possible absorption for your respective amount of blood loss, to stick with the maximum duration of use of 8 hours and to alternate between using tampons and sanitary pads.

WARNING!! nevernot soft tampons are not for contraceptive use and do not offer protection against sexually transmitted diseases. nevernot soft tampons must only be worn once. The repeated use of a tampon can lead to infection. Soft tampons are generally safe to use in combination with condoms or IUDs. If you are wearing an IUD, we recommend that you consult your gynaecologist before using soft tampons. We advise to not use soft tampons before your first penetrative intercourse.

Quality Standards: nevernot soft tampons are produced and sterilized to highest quality standards. This product is made of polyurethane foam with skin compatibility certified to Standard 100 by Oeko-Tex® and proven biocompatibility (acc. to DIN EN ISO 10993-5). nevernot soft tampons are free from products of animal origin. The packaging is recyclable. Please always store nevernot soft tampons in the original packaging and in a dark and dry place. Any possible discoloration of the product is due to the nature of the material used but does not impact the quality and usability of the soft tampon. Please note the expiration date on the packaging.

For further information please visit www.nevernot.de.

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