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East Midlands Pharma Ltd, Unit 1 Rothley Lodge,
Loughborough Road, Rothley, Leicestershire, LE7 7NL

EC REP Advena Ltd. Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

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Version 024-02

EN

Earwax or cerumen is a yellowish-brown wax-like substance produced by glands in the ear canal to help lubricate, protect, and clean the ear canal. Normally the wax and any trapped particles will move along the ear, to the opening of the ear where it is expelled. For some people this wax can build up which can lead to partial hearing loss or ear pain.

Earwax Removal drops contain glycerol to gently soften the wax, and urea peroxide, which releases oxygen to break up the hardened earwax deposits.

Ingredients

Glycerol BP, Urea Hydrogen Peroxide 5%

Do not use Earcare Removal drops if:

- Your ear drum has been damaged or perforated, you have an infection or any problems with your ears (eczema, itching sensation, dermatitis and any other local cutaneous (skin) problem).
- You are sensitive to any ingredients listed above.

Do not use this product after the expiry date printed on the carton and label.

Use within 30 days of first opening.

Warnings

- Children should receive medical diagnosis before using these drops to confirm earwax is present.
- Earcare removal drops should only be used in the ear.
- Avoid contact with the eyes, in case of accidental contact, rinse thoroughly with clean water and consult a doctor.
- Do not use if ear drum has been damaged or perforated, if there is an infection or any problems with ears (eczema, itching sensation, dermatitis or any other local cutaneous (skin) problem).
- Do not swallow.
- Do not use if you are sensitive to any of the ingredients.
- Do not use at the same time as any other ear drop preparations.
- Do not insert hearing aids immediately after use but allow at least 15 minutes for any residue to be removed.
- Do not use if seal on the bottle is broken before first use.
- Keep out of the sight and reach of children.
- For external use only.

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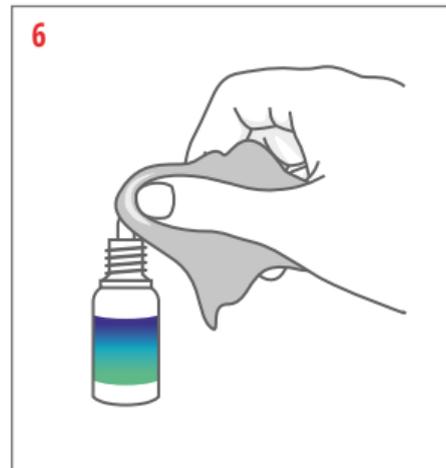
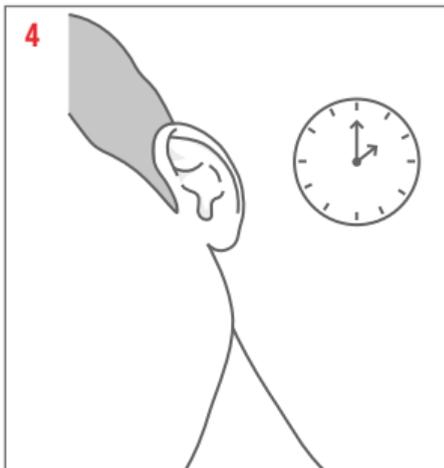
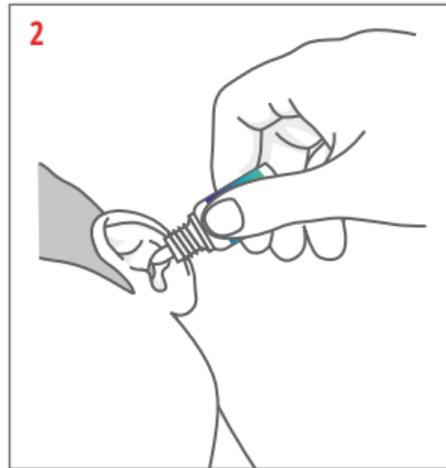
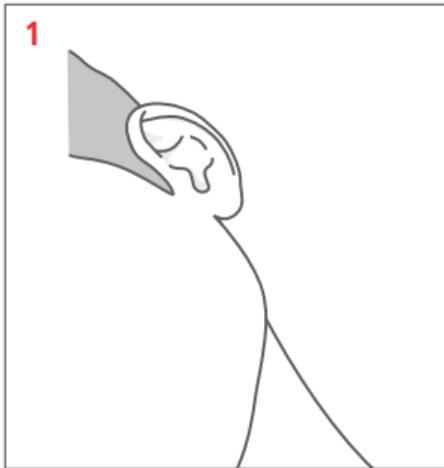
EN

Directions of use

You should follow the instructions to ensure removal of wax.

Check that the nozzle and bottle are not damaged or broken before use.

1. Tilt the head to one side with the affected ear facing upwards.
 2. Avoid touching the ear with the nozzle and do not put nozzle into the ear canal.
 3. Drop 3 to 6 drops into the ear and gently massage around the outside of the ear.
 4. Keep the drops in the ear for a few minutes.
 5. Then rinse with lukewarm water, to remove the remaining earwax.
 6. Gently wipe the nozzle with clean cotton wool after use.
- Repeat with other ear if required.
 - Use once or twice a day, for up to 6 days if necessary.
 - As the earwax begins to soften, before it is removed, it can begin to swell slightly, which might give a feeling of fullness in the ear, temporarily affect hearing, and occasionally, can cause slight dizziness or affect balance. A temporary bubbling sensation may occur. Should these symptoms persist or worsen, or if you experience any other side effects or burning, stop using Earcare Removal drops and seek advice from your doctor or pharmacist.

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