EN

Plaster/Adhesive bandage

Intended purpose: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the plaster/adhesive bandage. Remove plaster/adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster/adhesive bandage. Not suitable for children under 3 years old. In rare cases, a reaction to the plaster/adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster/adhesive bandage is used especially in the face. In both cases consult a doctor immediately. If you are diabetic consult your doctor before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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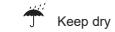
Do not reuse

Keep away from sunlight

Wrapper contains natural rubber latex

Do not use if package is damaged and consult instructions for use

Temperature limitation

22 PAP

N Distributed by: New Flag GmbH, Leopoldstrasse 154, D-80804 Munich, Germany. New Flag UK Ltd., Berkeley House, Amery Street, Alton, Hampshire, GU34 1HN, UK.

R RFX+CARE Manufacturing Co., Ltd. 7 Lanjiang Road, Yuecheng District, Shaoxing, Zhejiang, China 312000

C CMC Medical Devices GmbH Bahnhofstrasse 32, CH-6300 Zug, Switzerland

Contains CE-approved medical products

© Copyright. All rights reserved by RFX+Care International A/S NF Textile plaster-IFU 2023-10

175x210mm