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UK: BioBloom pain patch instructions for use

Please read these Instructions for use carefully before applying the Pain Patch.

Pain Patch is a registered medical device. Efficacy and safety have been tested according to the European Regulation for Medical Devices (2017/745).

What is Pain Patch and how does it work?

The BioBloom Pain Patch is a medical device that is applied externally to the skin in areas where joint or muscle pain is present. It offers two key effects: Firstly, it provides a rapid cooling effect by causing the solvents within the patch to evaporate quickly, leading to a cooling sensation on the treated skin. Secondly, the patch stimulates the nerves and muscles underneath, resulting in a prolonged pain relief effect. This stimulation improves blood flow to the muscles, aiding in long-lasting pain relief.

Intended use:

Produces a conductive cooling effect, and helps relieve muscle/joint pain.

Usage Benefit:

Relieves pain. Provides long-lasting pain relief. For muscles, neck, back, shoulder, and joint pain, sprains, sports injuries, and arthritis.

Application:

Apply the patch to the area you wish to treat on clean and dry skin. Tear-open the sachet and reveal the pain patch. Detach the patch from the protector film and place it on the painful area. Use only once a day, do not keep the patch on for more than 24 hours. Avoid applying the patch on injured or irritated skin, such as cuts, scrapes, sunburns, infections, or rashes. Follow all instructions on the product packaging. The product is intended for temporary use.

Ingredients:

Active ingredient: 94,4% Acrylic adhesive
Excipients: cannabidiol, menthol and arnica fluid extract
Pain Patch contains alcohol.

Dosage form and content:

Original package with 12 patches.

Warnings:

- For topical use only.

- Do not use this product on damaged or injured skin.
- Avoid eye contact. Wash hands carefully after use.
- Keep out of reach of infants and children.
- Keep out of reach of pets.
- Not recommended for children under 14 years of age - unless consulted with your doctor.
- Persons with an allergy to alcohol, solvents, or any of the ingredients should not use the product.
- During pregnancy or breastfeeding, use only after consulting a doctor.
- If there is no improvement 2 - 3 days after starting the treatment or if the symptoms have worsened considerably, please consult a doctor.

Drug interaction:

To avoid interactions, apply medicines two hours apart or consult your doctor.

Side effect:

No side effects are known when used as directed.

Contraindication:

Do not use the BioBloom Pain Patch in case of hypersensitivity (allergy) to any of the ingredients.

Storage instructions:

- Store in a cool (between 10°C and 25°C) and light-protected place.
- Tightly close the sachet after use.
- The Pain Patch may only be used until the date marked „use by“ on the box/sachet. If the stated expiry date has passed, the product must no longer be used.
- Due to the nature of the ingredients, a slight pink tinge may appear after some time. This does not affect the quality of the product.

Disposal:

The product can be disposed of with normal household waste.

Incident reporting:

If this product is associated with a serious incident (death of a patient, user or other people; temporary or permanent serious deterioration of the health status of a patient, user or other people; serious risk to public health), this incident must be reported to the manufacturer and the competent national authorities.

pain patch

BioBloom CBD

Distributor: BioBloom GmbH,
Frauenkirchnerstr. 12,
7143 Apetlon, Austria,
office@biobloom.at,
www.biobloom.com

Manufacturer: CANNORDIC A/S,
Indiakaj 10, 2100 Copenhagen,
Denmark

Symbol CE

Temperature limit

Use-by date

Consult instruction for use

Batch code

Medical device

Keep away from sunlight

Do not use if the packaging is damaged

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Distributeur: BioBloom GmbH,
Fabricant: CANNORDIC A/S,