DE

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EN

TRUETAPE is an elastic, breathable, and water-resistant kinesiology tape that is made from cotton (TRUETAPE® BASIC) or a spandex material (TRUETAPE®) and uses a hypoallergenic acrylic adhesive. It offers you support during regeneration after exercise or rehabilitation after injuries.* TRUETAPE is a premium kinesiology tape and can be easily applied at home with the help of our taping instructions.

Material TRUETAPE®: 96% viscose | 4% elastane | hypoallergenic acrylic adhesive
Material TRUETAPE® Basic: 96% cotton | 4% elastane | hypoallergenic acrylic adhesive

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EN

Thank you for choosing TRUETAPE. We hope to help you and your body!

OUR APPLICATION TIPS

Only apply TRUETAPE according to instructions and please follow these simple steps to get the best taping result.

Apply tape 1 hour before starting an activity.

Thoroughly remove dirt, moisture, oils and lotions from the skin before taping.

In areas that are particularly hairy, trim the hair to ensure the tape sticks better.

Pay attention to the information on stretching in the PDF and video instructions.

Remove backing film before sticking and always make sure the ends of tape are not under tension when applied.

To activate the adhesive, carefully rub the tape several times after sticking the tape down.

The tape is waterproof. After showering, bathing, or swimming, simply pat the tape dry using a towel.

Do not wear the tape for longer than 7 days.

Do not rip the tape off the skin in one sudden movement; rather, you should gently peel it off in the direction of hair growth.

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HEEL PAIN

AND MANY MORE.

All TRUETAPE video and photo tutorials can be found at www.truetape.eu

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***WARNING:**

Do not use TRUETAPE on injured or irritated skin. If redness, irritation, pain or itching occurs, carefully remove it from the skin. The effect of kinesiology tapes has not been scientifically proven. Our statements are based on many years of experience and application reports from patients and therapists. TRUETAPE and the information and instructions we provide are not a substitute for professional medical treatment and are not suitable for self-diagnosis. In pregnancy, use only in consultation with your doctor. Keep out of the reach of small children. Do not use if the plastic cover is damaged. Do not use if expiration date has passed.

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