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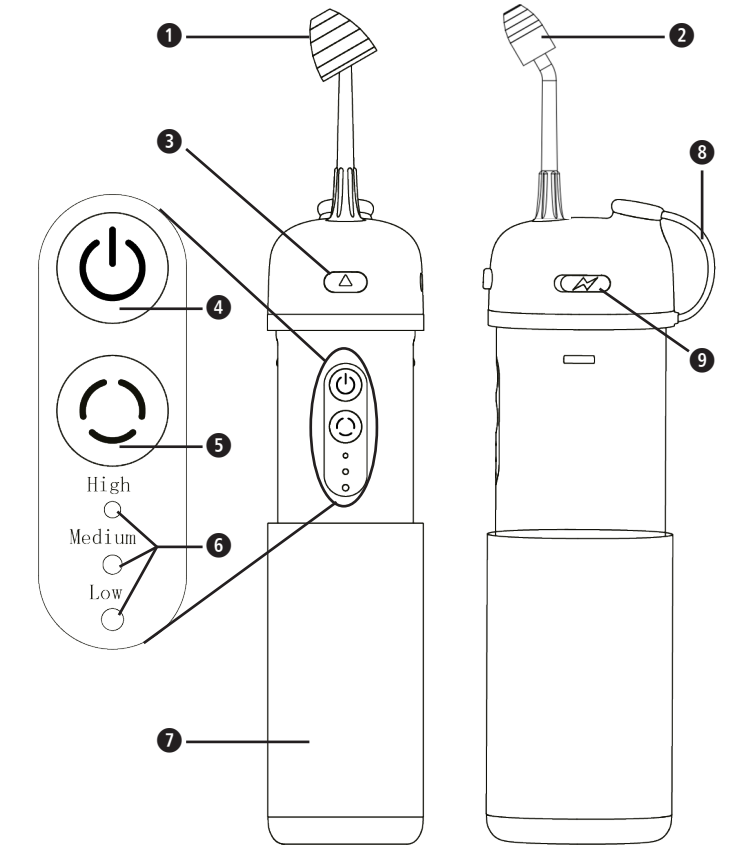
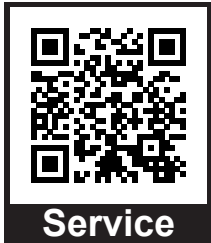
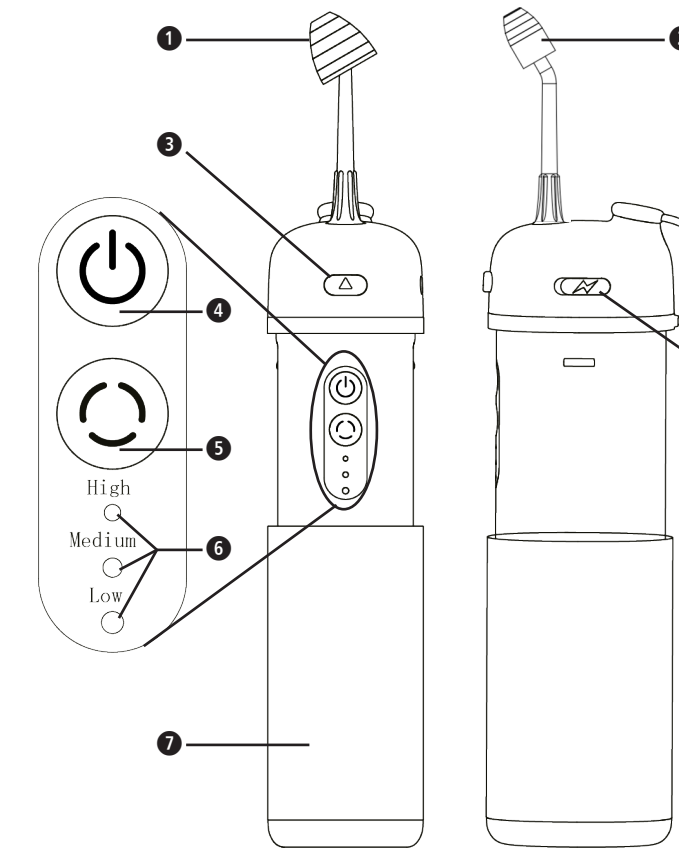
PDF »

Device and controls

- 1 Nasal water jet attachment (for adults)
- 2 Nasal water spray attachment (for children and adults)
- 3 Spray head lock button
- 4 On/off button
- 5 Mode button
- 6 Working level
- 7 Reservoir
- 8 Cover lanyard
- 9 Charging port

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warnings must be observed to prevent possible injury to the user.

ATTENTION
These instructions must be followed to prevent possible damage to the appliance.

NOTE
These notes provide you with useful additional information on installation or operation.

LOT LOT number

Manufacturer

Recycling symbols/codes:
These are used to provide information about the material and its proper use and recycling.

Scope of delivery and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your dealer or your service centre. The following parts are included:

- 1 **medisana** Nasal Irrigator **NI 200**
- 1 Charging cable
- 1 Nasal water jet attachment (for adults)
- 1 Nasal water spray attachment (for children and adults)
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING
Make sure that the packaging film does not get into the hands of children. There is a risk of suffocation!

IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Safety Information

- When using an electronic appliance (especially if children are present), basic precautions should always be followed, including the following.
- Use the device only according to its intended purpose as specified in the instruction manual. The warranty will be invalidated if used for purposes other than those for which it is intended.
- Any application other than the herewith described is improper and therefore considered dangerous. If the appliance is not used according to the instructions, the user is liable for the safe operation of the unit.
- Oral/Nasal irrigators can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge under supervision.
- The use of the appliance on children under 8 years of age is not recommended. Children aged 8 and over must be supervised by an adult.
- It is also recommended that adults first learn the standardised use of nasal showers before instructing children in their use.
- We recommend that children only ever use the lowest working level 'low'.

Caution! Reduction of the risk of electric shock:

- Do not use when bathing.
- Never store the appliance in such a way that it can fall or be pulled into a bath, shower cubicle or sink.
- Do not place or drop in water or other liquids.
- Do not reach for an appliance that has fallen into water. Unplug the appliance immediately.
- Do not touch the charger with wet hands when charging.

Warning! To reduce the risk of burns, electric shock, fire or injury:

- The appliance should never be left unattended.
- Close supervision is required when this product is used by, on or near children or disabled persons.
- Use the appliance only for the purpose described in this manual. Do not use attachments that are not recommended by the manufacturer.
- Do not operate the appliance if it is damaged, if it is not working properly, if it has been dropped or damaged or if it has fallen into water. Take the appliance to a service centre for inspection and repair.
- Keep the cable away from hot surfaces.
- Never block the air openings of the appliance and do not place it on a soft surface, such as a bed or couch, where the air openings could be blocked. Keep the air openings free of fluff, hair and the like.
- Never use the appliance when you are asleep or drowsy.
- Do not drop or insert any objects into the openings.
- Disconnect the charging cable before filling the appliance. Only fill the device (reservoir) with water or a medical saline solution, unless otherwise specified by the manufacturer. Do not overfill!
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- Do not use the device outdoors or in places where aerosol products (sprays) are used or where oxygen is administered.
- The appliance may only be operated with safety extra-low voltage in accordance with the labelling on the appliance.
- Do not connect this device to a voltage system other than that specified on the device or charger.
- This device contains batteries (rechargeable battery) which are not replaceable.
- The device is only suitable for domestic use. Do NOT use the product outdoors.
- People with respiratory diseases or after sinus surgery should refrain from using the device unless expressly recommended by a doctor.

Warning! To reduce the risk of electric shock and other hazards, please observe the following safety guidelines:

- Check whether the charger or the cable is damaged before you start up or continue to use the device. If either is damaged, do not attempt to use the device. Instead, contact customer service for a replacement.
- Do not throw the appliance into a fire or other hot environment.
- Do not immerse the device in water as this may cause damage.
- Do not attempt to plug or unplug the charger with wet hands.
- Do not modify, damage or destroy the device. Do not pull, bend or twist the cable by force. Do not place heavy objects on the cable and do not clamp the cable between other objects.
- Do not disassemble the device, except to remove the battery for disposal (see corresponding chapter).
- Do not place the device, the charger or the cable on heated surfaces.
- Do not place the device in a hot or humid place or in a place exposed to direct sunlight.
- Before cleaning and maintenance, ensure that the device is switched off and is not charging.

Usage: Charging the battery

We recommend fully charging the **medisana NI 200** nasal irrigator before using it for the first time (maximum charging duration: approx. 2.5 hours). The device cannot be used during the charging process. To charge, connect the supplied cable to the charging port and to a 5V /1A power source. During the charging process, the 'Low' LED of the working level indicator flashes. When the battery is fully charged, all 3 LEDs of the power level indicator light up constantly for approx. 20 minutes.

If the battery charge level is low during operation, the currently active LED of the working level indicator flashes - charge the battery as soon as possible. If the battery charge level is so low that further use is not possible, the 'Low' LED on the working level indicator flashes 3 times in quick succession before the appliance is automatically switched off. The appliance cannot be used again until the battery has been recharged.

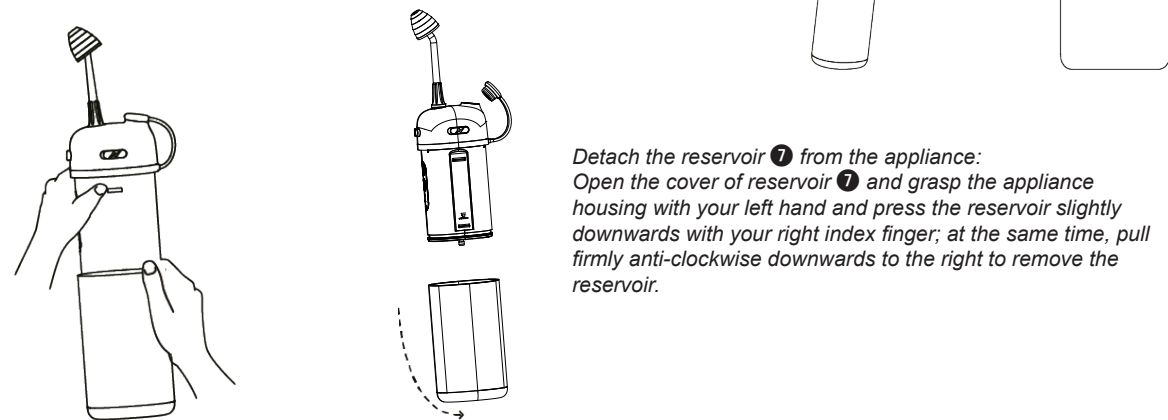
Installing the spray nozzle

Insert the nasal water jet attachment (for adults) or the nasal water spray attachment (for children and adults) into the centre hole of the appliance head until it clicks into place.

Filling the reservoir

Open the cover of reservoir and fill it with water or isotonic saline solution. Alternatively, you can carefully pull the reservoir downwards from the device (see drawing below) to fill it more quickly.

Note: Make sure that the device is switched off when filling reservoir. Then close the reservoir again or reconnect the reservoir to the device.

Detach the reservoir from the appliance: Open the cover of reservoir and grasp the appliance housing with your left hand and press the reservoir slightly downwards with your right index finger; at the same time, pull firmly anti-clockwise downwards to the right to remove the reservoir.

Use

When the appliance is switched off, press and hold the On/Off button to switch it on. When switched on, briefly pressing the On/Off button switches the appliance off. After switching on, the appliance switches to standard operating mode.

Lean over a washbasin, open your mouth to breathe and slowly place the spray nozzle on your nostril (make sure that you do not insert the nozzle tip directly into your nostril). When both nostrils are clear, you can slowly close one nostril with the nozzle tip to allow the saline solution or water to flow back out of the other nostril or mouth, cleaning the entire nasal cavity and deep sinuses.

Briefly press the mode button to switch between high, medium and low operating levels. If you are using the appliance for the first time, we recommend that you start at the lowest working level. You can increase this later depending on your personal preference. The working level is indicated by the respective LED.

If you have nasal congestion on one or both sides, do not block the nostrils directly with the spray nozzle. Slowly guide the nozzle tip towards one nostril and try to insert the tip further into the nostril after a while. If liquid flows out of the other nostril or the mouth, this means that the nasal cavity has become clearer.

At this point, you can rinse your entire nose thoroughly. If you feel increased pressure in the nasal cavity or middle ear and the fluid cannot drain out of the other nostril or through the mouth, remove the spray nozzle and try the above steps again later.

Note: If the water or saline solution does not come out of the other nostril or mouth even after repeated attempts, stop using the device to prevent the excessive pressure in the nasal cavity from causing you harm.

If you have inserted the spray nozzle into your left nostril, tilt your head to the right so that the liquid can flow easily out of the right nostril and vice versa. When using the device, do not speak, keep your mouth open for breathing during the entire procedure and do not swallow saliva.

Warning: Always pay attention to the pressure in the nasal cavity and middle ear. If your nose is blocked, the flow of water from the nasal irrigator can cause excessive pressure in the nasal cavity and middle ear, resulting in discomfort. If you feel discomfort or pain, stop using it immediately.

Do not blow your nose immediately within 5 minutes of using the device. Use tissues to gently push the water out of the nasal cavity; otherwise, discomfort may occur due to increased pressure in the middle ear. Do not use the device continuously for longer than 2 minutes per application. The device switches off automatically after approx. 2 minutes of continuous use.

Use on children

We strongly advise against the use of the appliance by children under 8 years of age. Children aged 8 years or older must use the device under adult supervision. Children are still growing and their nasal cavities are not yet fully developed. They have shorter and smaller nasal cavities than adults. Therefore, the use of the device is different for adults and children. Adults should instruct their children to use the device correctly and safely and the duration of use should be shorter than for adults. We recommend that children only ever use the lowest working level ('low').

After use

Switch off the appliance, open the reservoir with the water inlet opening towards the bottom and pour out the remaining liquid as shown in the illustration. Switch on the appliance and allow the remaining liquid to drain until the spray nozzle no longer sprays any liquid and then switch off the appliance.

Removal of the spray nozzle

If you want to remove or replace the spray nozzle, you can do this by pressing the spray head lock button and pulling out the spray nozzle at the same time.

Cleaning and care

Cleaning the appliance:

- Only clean the appliance with clean water or a neutral cleaning agent. Do not use any corrosive or abrasive cleaning agents.

- Never place the appliance in water or liquids. If necessary, clean the surface of the appliance with a soft cloth dipped in a mild, non-corrosive cleaning agent.

Cleaning the reservoir:

- Hold the device with your left hand and the reservoir with your right hand. Pull the reservoir downwards to separate it from the appliance.
- Rinse the inside of the reservoir with clean water at room temperature. Do not clean the reservoir with additional cleaning agents.
- After cleaning, put the reservoir back in its original position.

Removal of mineral deposits caused by hard water:

Depending on the mineral content of the water used, deposits may form on the appliance. If you do not remove deposits from hard water, the performance of the appliance will be impaired. We therefore recommend repeating the following cleaning process every 1-3 months to maintain the performance of the appliance:

Point the spray nozzle at a sink, switch on the appliance and let it run until all the water or saline solution has been used up. Fill reservoir with clean, hot water (>85°C) and then repeat the above procedure.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Troubleshooting

Problem	Possible cause	Solution
Device is leaking (e.g. leak between spray nozzle and device body or in reservoir)	The seal is damaged or the spray nozzle has not been installed in the correct position.	Install the spray nozzle or the reservoir in the correct position.
Insufficient spray pressure	Battery not sufficiently charged.	Charge the battery.
Device does not work	Battery not sufficiently charged.	Charge the battery.
Device is working, but no liquid comes out of the spray nozzle	Device is being used for the first time or has been used for a long time.	Fill the reservoir to the maximum and allow the appliance to operate at the highest level for a few seconds.

If the error cannot be rectified using the table, please contact customer service.

Technical specifications

Name and model:	medisana Nasal Irrigator NI 200
Power supply:	3.7V= 1100mAh / Input: 5V= / 1A 5W (via USB)
Power consumption:	8W
Flow rate:	1 Nasal water jet attachment (for adults) „low“: 155ml±30ml/1min / „medium“: 220ml±40ml/1min / „high“: 265ml±50ml/1min 2 Nasal water spray attachment (for children and adults) „low“: 40ml±20ml/1min / „medium“: 50ml±20ml/1min / „high“: 60ml±20ml/1min IPX7
Water resistance:	2.5 hours
Charging time approx.:	77.7 x 48.7 x 141 mm
Dimensions approx.:	54595
Article no.:	40 15588 54595 5
EAN code:	Art. 54596 (EAN code 40 15588 54596 2): Nasal Kit, consisting of: 1 x Nasal water jet nozzle (for adults) 1 x Nasal water spray nozzle (for children and adults) 2 x Tips for nasal water jet nozzle 2 x Tips for nasal water spray nozzle
Spare parts:	

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.

2. Defects in material or workmanship will be removed free of charge within the warranty period.

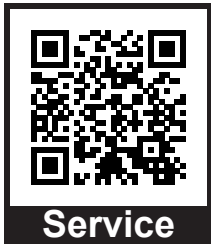
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.

4. The following is excluded under the warranty:
a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
d. Accessories which are subject to normal wear and tear.

5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

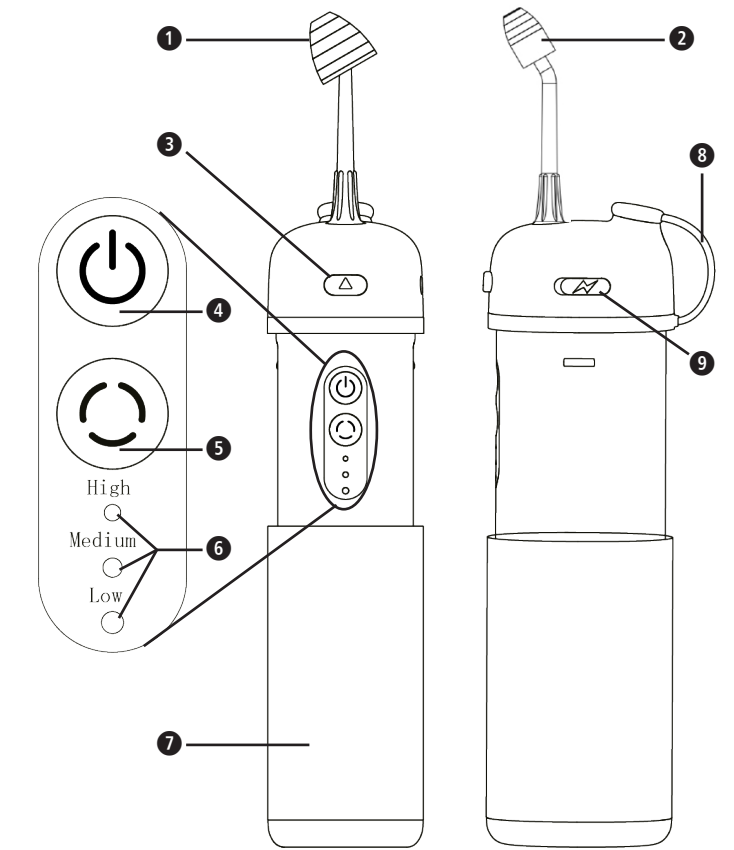
Service information is available here: <https://www.medisana.com/servicepartners>

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