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The world’s first vegan pregnancy test

VEGAN

CRUELTY
FREE

DAYS EARLIER

APP-CHECK
+ ESTIMATED TIME
OF FERTILISATION

Test tip with protective cap
Control screen with control line
Test screen with test line
Code for the digital result
Handle

Intended use

The hey mela pregnancy test is an immunological rapid test for the semi-quantitative detection of human chorionic gonadotropin (hCG) in the urine of persons of childbearing potential. The test is intended for self-testing and serves as an aid for the early detection of pregnancy. The test result can be evaluated using the hey mela app. Alternatively, a visual evaluation can be carried out by the person performing the test based on the presence of both lines: a control line and - in the case of pregnancy - a test line. If both lines are visible, the intensity of the test line can be evaluated in comparison to the control line: A weaker test line indicates a time of fertilisation before 1-2 weeks, an equally strong or stronger test line indicates a time of fertilisation longer than 2 weeks. If the test line does not form, the result is negative and there is no pregnancy, and the result obtained should always be discussed with a doctor. The test is an in-vitro diagnostic for single use and is not intended for near-patient testing. It is not an automated test system. The test is intended for over-the-counter dispensing.

Test principle

The hey mela pregnancy test supports the early detection of pregnancy by detecting hCG. Various antibodies are used for this purpose, which enable selective detection of hCG in a urine sample. After the sample is applied, the sample migrates over the test strip by capillary action and reacts with the colored antibody conjugate, which indicates the test result by forming a test and/or control line.

Antibodies used, recombinantly produced in diatoms: 0.375 µg gold-labeled anti-hCG antibody, 1.6 µg membrane-fixed anti-hCG antibody, 0.6 µg membrane-fixed control antibody.

Materials

included: hey mela pregnancy test, instructions for use
not included: clean container, timer, smartphone, hey mela app

Expected values

1. Negative results are expected in healthy, non-pregnant persons.
2. Positive results are expected in pregnant persons, although the amount of hCG varies from person to person.

Questions & Answers

How does the hey mela pregnancy test work?

After fertilisation, the placenta begins to produce the pregnancy hormone hCG, which is released into the urine. The hCG level rises continuously and reaches its maximum value by the third month of pregnancy. The hey mela pregnancy test detects this hormone with a sensitivity of 10 mIU/mL.

When can the test be used?

The hey mela pregnancy test can be used up to 4 days before your period is due. Since the hormone level is individual, a negative result may still appear if the value is low.

Can taking medication affect the test result?

Alcohol, medication such as painkillers and antibiotics as well as the contraceptive pill should not influence the test result. Read the instructions for your medication before taking the test.

Specificity

The test was determined by cross-reactivity studies with known amounts of luteinizing hormone (LH), follicle-stimulating hormone (FSH) and thyroid-stimulating hormone (TSH). The addition of LH (500 mIU/ml), FSH (1,000 mIU/ml) and TSH (1,000 µIU/ml) to hCG-negative and hCG-positive samples showed no cross-reactivity. The hCG-beta-core-fragment, which is only present later in the pregnancy, is detected. Up to a test concentration of 500 pmol/ml, no influence on the test result was observed.

Sensitivity

Sensitivity was confirmed with urine hCG standards. The results showed that the analytical sensitivity (the lowest concentration that leads to 95 % positive results) is 10 mIU/ml (exactly 8.18 mIU/ml).


Accuracy

In a method comparison study (with a CE-marked reference device) with urine samples (number of urine samples positive = 250, negative = 100), an overall agreement with a hCG urine test of 100 % (95 % confidence interval: 98.95 % - 100 %) was determined. The following data were determined:

Days before missing your period	-1	-2	-3	-4
Proportion of positive tests with the result "pregnant; 1-2 weeks"	> 99 %	> 99 %	> 99 %	> 66 %

Use with hey mela app

- Download the hey mela app from the Google Play Store or the Apple App Store and start the app.
- Now follow the instructions in the app:

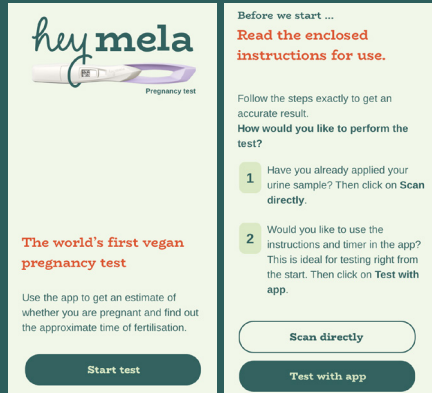- When you select "Test with app", all the steps for carrying out the test correctly are displayed. You can then start the timer (6 min) and after the time has elapsed you can scan your test.

- A positioning window will open to scan the test, after which your result will be displayed.

Use without the hey mela app

1. Open the foil pouch immediately before performing the test.
2. Remove the test stick and pull off the protective cap. Avoid touching the test tip.
3. You have two options for carrying out the test, always hold the test with the handle pointing upwards and the test tip pointing downwards. Make sure that only the tip and not the rest of the housing comes into contact with the urine sample.
 - **Directly in the urine:** Hold the test tip with the absorbent side down in your urine stream for approx. **5 seconds**.
 - **With a urine sample:** Collect your urine in a clean, dry container. Immerse the test tip completely in the urine sample for approx. **10 seconds**.
4. Replace the protective cap and place the test on a flat, horizontal surface.
5. A positive result may be visible after just 1 minute from the absence of your period. For an assessment of the time of fertilisation or in the case of an early test, read between **6 and 15 minutes**. Always wait until the red control line appears.

Results after 6 – 15 minutes

The app evaluates the test and control lines with the following result

Estimated time of fertilisation
1-2 weeks

Estimated time of fertilisation
No pregnancy identifiable

Estimated time of fertilisation
more than 2 weeks

If one of the following messages appears, repeat the test with a different test stick.

- No lines visible: New test detected
Your test is either new or incorrect. If you have already performed the test, please contact support.
- No control line: Invalid result
The test stick is invalid because no control line was detected.

Possible warnings during the scan process

- Overexposure detected
- Reflections detected
- Scan time expired
- Detection error
- Batch recalled
- Batch expired

Reposition your camera and follow the descriptions of the notes.

Notes

- This test is only suitable for self-administration.
- The test is not intended for monitoring the progress of a pregnancy.
- Perform the test **before your period is due, preferably with morning urine**.
- Only use fresh urine to perform the test.
- Inspect the foil packet for damage before performing the test. Do not use damaged or opened foil packets.
- Check the printed expiry date and do not perform the test if it has already been reached or exceeded. Damaged and opened foil packs should not be used.
- After opening the foil pouch, the test should be used within one hour.
- The hey mela app is compatible with all smartphones with iOS 15.0 or higher or Android 8.0 or higher. The app works from software version 1.0.0. An integrated camera with at least 720p (HD) resolution and a functioning camera light (flashlight) are required. Please note: The app does not work reliably on devices with a foldable display, as display problems may occur. Therefore, use a classic smartphone model.
- Read the test result yourself when using the hey mela app.
- Keep the test out of the reach of children.
- Dispose of the used test in household waste.
- These instructions for use are also available digitally at www.heimela.de.

Explanation of symbols

	Sufficient for <n> tests		Do not reuse
	If packaging is damaged, do not use		Temperature limit (store between 2-30 °C)
	Store in a dry place		Protect from sunlight
	In vitro diagnostic medical device		Unique Device Identifier
	Article no.: 2.13.1.715, 2.13.1.722, 2.12.1.723		Batch code
	Best before date		Observe instructions for use
	Manufacturer		European conformity

Significant additions or changes are marked in the margin. We ask you to report all (serious) incidents related to the hey mela pregnancy test to VELLAP Diagnostics GmbH and the competent authority of the Member State in which you are established. © 2025, VELLAP Diagnostics GmbH

10.07.2025, Rev. 1.3.5.1936-3

Interference study: The following potentially interfering substances were added to hCG-negative and positive samples.

Substance	concentration [mg/dl] / parameter	Substance	concentration [mg/dl] / parameter
Unconjugated bilirubin	60	Salicylic acid	2.86
Nitrite	1.5	Ampicillin	33
Leukocytes	1.5 x 10 ⁹ cells/l	Acetaminophen (paracetamol)	7.8
Albumin	5300	Ethanol	1000
Glucose	2000	Levonorgestrel	45
Hemoglobin	1000	Ethinylestradiol	10
pH value	4 to 9	Pregnanediol glucuronide	20
Aspirin (acetyl-salicylic acid)	43.5		

potentially cross-reactive substance	concentration	potentially cross-reactive substance	concentration
luteinizing hormone (LH)	500 mIU/ml	thyroid-stimulating hormone (TSH)	1,000 µIU/ml
follicle-stimulating hormone (FSH)	1,000 mIU/ml	hCG-beta-core-fragment	500 pmol/ml

At the concentrations tested, none of these substances affected the expected test results.

Threshold value for estimating the time of fertilisation: The median reference urine hCG value for distinguishing weeks 1-2 and week 2+ (in relation to ovulation) in the presence of pregnancy is 137 mIU/ml and was confirmed using urine hCG standards with an overall agreement of 100 % to a CE-marked reference test.

Precision: The precision study showed 96 % agreement between the results obtained under controlled conditions with three product batches.

High-dose hook: The effect of high concentrations of hCG on the test was evaluated using hCG standards with a concentration of up to 400,000 mIU/ml. The results show that this product does not exhibit a hook effect. High concentrations of the hCG degradation product hCG-beta-core-fragment of up to 500 pmol/ml also show no hook effect.

Standardization: The test was standardized according to the international standard of the WHO, the 6th WHO chorionic gonadotropin (NIBSC code: 18/244).

VELLAP Diagnostics GmbH
Industriestraße 8
99427 Weimar, Germany

Help & Contact:
hello@heimela.de