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DE	Traubenkern-Kissen mit Bezug – Gebrauchsanweisung.....	2
EN	Grape Seed Pillow with Cover – Instructions for Use.....	7
FR	Coussin aux pépins de raisin avec housse – Notice d'utilisation.....	12
IT	Cuscino con vinaccioli con fodera – Istruzioni per l'uso	17
NL	Druivenpitkussen met hoes – Gebruiksaanwijzing.....	22
ES	Cojín de pepitas de uva con funda – Instrucciones de uso	27

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Mikrowelle

Backofen

EN Grape Seed Neck Pillow with Cover - Instructions for Use

Dear Customer,

Below you will find important information about this product. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See packaging!

1. Scope of Application

These instructions for use are applicable for the following products:

- REF** 185-10 Grape Seed Neck Pillow with Cover
186-10 Grape Seed Heat Pillow for the Back and Stomach with Cover

2. Contents

Grape Seed Neck Pillow with Cover or Grape Seed Heat Pillow for the Back and Stomach with Cover, Instructions for Use

3. Intended Purpose and Use

The Grape Seed Neck Pillow can be used to apply momentary heat to increase feelings of wellbeing and especially for easing the body's aches and pains as a form of heat therapy. The product is only intended for private home use and is only suitable for adults and children over the age of 14.

4. Areas of Application

Grape Seed Neck Pillow with Cover

The Grape Seed Neck Pillow is suitable as a warmer exclusively for the following areas of application:

- Muscle pains/stiffness
- Muscle/neck tensions
- Generally warming cold areas of the body

Grape Seed Heat Pillow for the Back and Stomach with Cover

The Grape Seed Heat Pillow for the Back and Stomach is suitable as a warmer exclusively for the following areas of application:

- Muscle pains/stiffness
- Stomach ache/Stomach cramps
- Constipation and flatulence
- Discomfort during menstruation
- Generally warming cold areas of the body

5. Usage and Heating Instructions

Before heating the pillow, check for any unusual odors, mould stains or burn marks. Dispose of the product immediately if these are present. Please remove the inner pillow from the cover before heating. The following options are available to heat the pillow:

- Warm in a non-preheated oven at a max. temperature of 110 °C for max. 5 minutes
- Warm in the microwave at max. 300 watt for max. 2 minutes (Follow the instructions of the microwave)

manufacturer!) If you don't think the product feels warm enough, extend the heating time in 15 second intervals at the specified wattage, under personal supervision, up to a maximum temperature of 48 °C.

- Place on a lukewarm radiator or tiled stove for a longer period of time

Immediately after heating the pillow, distribute the heat evenly by shaking it! Subsequently insert the inner pillow into the cover. When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

6. CAUTION! Important Safety Information

It is essential that you strictly follow the heating instructions on the product label and in the above heating guidelines for your product, as longer heating times or higher energy levels may cause damage (risk of inflammation and burning)! Do not heat up the cover! Before heating the pillow in the microwave, check that heating grain products is permitted in the manufacturer's instructions for use. Only under adult supervision and never heat when wet! Let the pillow cool down before reheating it!

The product should be used without direct skin contact on a thin textile close to the body and should never be used as a pillow or seat. If the symptoms persist or worsen, please seek advice from a doctor. Do not use the pillow if you suffer from an acute, chronic, inflammatory disease or an illness accompanied by a temperature, circulatory

problems, fresh hematomas, bruises and open wounds or for individuals with a reduced sensation of pain. If there are existing skin conditions, heart disease, venous disease, diabetes or tumor disease, or if the user is pregnant please contact a doctor before use. Pay attention to intact seams! Loose grape seeds are a choking and suffocation hazard! Only use under direct supervision of an adult!

If the seams have come open and loose seeds are leaking out, dispose of the product and the loose seeds immediately and make sure they are out of reach of children.

All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

7. Care Advice and Storage

The cover can be removed and is machine-washable at 30 °C. Please observe the care symbols on the product! Do not wash the inner pillow filled with grape seeds in order to prevent the formation of rot/mould! Store the product in a dry place! Protect from moisture!

8. Information about Shelf Life and Disposal

With normal usage, the product should be replaced after 4 years. The product can be disposed of in household waste and recycled thermally without any problems. Please separate the individual packaging components and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

9. Material Composition

REF	185-10 Grape Seed Neck Pillow with Cover 186-10 Grape Seed Heat Pillow for the Back and Stomach with Cover
Fabric:	Cover: 100% polyester Internal pillow: 100% cotton
Filling:	Grape seeds

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